How to use your new Rehband Support - 7722

RU

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UK

Function

Takes the strain off the irritated tendon attachment and provides pain-relief. Conveys warmth to the affected elbow area. Permits movement in spite of inflammation and pain.

Indication

Tennis and golf elbow, mouse arm. Overexertion of the dorsal muscle groups in the lower arm/wrist.

Material

5 mm SBR/неопрен, polyamide jersey.

Application

- 1) Apply protection to the elbow. Adjust in the area of the upper arm.
- 2) The pelottes are positioned approx. 5 cm below the elbow joint on the injured muscle.
- 3) Pull tight enough to leave enough room for one finger under the tape and close the Velcro fastening.

Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) Re: SBR/неопrene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 6) If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 7) If you are allergic to heat, we advise you to use a product that not contains SBR/неопrene.

*Re: Medical- products, the prescriber must be a qualified doctor, PT, OT or Orthopaedic Engineer.

Washing Instructions

- 1) Close all Velcro.
- 2) Remove detachable splints.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry.

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